

cobas Lipid Control

cobas®

REF 06380182190

CONTENT 4 x 2.0 mL

SYSTEM cobas b 101

English

Intended use

cobas Lipid Control solution is used for performing quality control of total cholesterol (TC), high density lipoprotein (HDL), cholesterol and triglycerides (TG) with **cobas Lipid Panel** test on the **cobas b 101** instrument.

Summary

cobas Lipid Control is a ready-to-use solution based on human serum. The control is used for monitoring accuracy and precision of the **cobas Lipid Panel** test. The adjusted concentrations of the control components are below the thresholds (Level 1) or above the thresholds (Level 2) for classifying the lipid status.

Reagents - working solutions

Reactive components in the **cobas Lipid Control**:

- Human serum-based solution containing constituents of human origin
- Preservatives and stabilizers

Level 1, 2 bottles 2 mL each, below threshold

Level 2, 2 bottles 2 mL each, above threshold

Target values and ranges

The target values were determined with **cobas b 101** instruments using at least 3 lots of **cobas Lipid Panel**. Determinations were performed under strictly standardized conditions on the **cobas b 101** instrument. The target value is the mean of all values obtained. The corresponding control range is calculated as the target value ± 3 standard deviations (with the standard deviation being the value obtained from several target value determinations). Results must be within the specified ranges. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits. The concentrations of the components are lot-specific. The exact values are given in the enclosed value sheet. Target values and ranges for the **cobas b 101** instrument are also encoded on the QC info disc which is provided in the **cobas Lipid Control** kit.

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

All human material should be considered potentially infectious. All products derived from human blood are prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg and antibodies to HCV and HIV. The testing methods used assays approved by the FDA or cleared in compliance with the European Directive 98/79/EC, Annex II, List A.

However, as no testing method can rule out the potential risk of infection with absolute certainty, the material should be handled with the same level of care as a patient specimen. In the event of exposure, the directives of the responsible health authorities should be followed.^{1,2}

Reagent handling

The control is supplied ready for use. Gently mix the contents of each bottle before sampling to ensure homogeneity. Avoid foam formation. Treat control material in the same way as samples. Gently squeeze the bottle to form a small drop of solution at the tip. Carefully transfer the drop of solution onto the indicated blue application zone on the back side of the **cobas Lipid Panel** disc. The drop of solution is automatically absorbed, filling the application zone. Dispense the control liquid carefully without touching any other surface to avoid microbial contamination. Replace cap immediately. Proceed as described in the operator's manual.

When the QC info disc is used, the **cobas b 101** instrument automatically indicates the control ranges of this specific control lot. The **cobas b 101** instrument can be set up to show "Pass" or "Fail" on the display. The display shows "Pass" if the QC results are within the control range, and "Fail" if the QC results are out of the control range. For more details, please refer to the operator's manual of the **cobas b 101** instrument.

Storage and stability

Store at 2-8 °C.

Do not use **cobas Lipid Control** after the stated expiration date. Discard the bottle if there is excessive turbidity in the product or evidence of microbial contamination. **cobas Lipid Control** is a stabilized liquid product.

Stability

unopened:	up to the stated expiration date at 2-8 °C
after opening:	7 days at 20-25 °C or 30 days at 2-8 °C provided that the dispensing of the control occurs without microbial contamination and when stored tightly capped

Materials provided

- 2 x 2 mL **cobas Lipid Control** Level 1 (below threshold)
- 2 x 2 mL **cobas Lipid Control** Level 2 (above threshold)
- 1 x QC info disc

Materials required (but not provided)

- REF 06380115190, **cobas Lipid Panel**
- REF 06378668190, **cobas b 101** instrument
- General laboratory equipment

Assay

Accurate and reproducible results are dependent upon properly functioning instruments and reagents.

The control intervals and limits should be adapted to each laboratory's individual requirements. Values obtained should fall within the defined limits. Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

Follow the applicable government regulations and local guidelines for quality control.

References

- Occupational Safety and Health Standards: Bloodborne pathogens. (29 CFR Part 1910.1030). Fed. Register.
- Directive 2000/54/EC of the European Parliament and Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work.

For further information, please refer to the appropriate operator's manual for the instrument concerned, and the Method Sheets of all necessary components.

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard (for USA: see dialog. Roche.com for definition of symbols used):

CONTENT

Contents of kit

SYSTEM

Analyzers/Instruments on which reagents can be used

COBAS, COBAS B are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2019, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www. Roche.com

