# **CoaguChek<sup>®</sup> INRange System**

## *Supplement to the User's Manual version 2.0*

Changes from SW 01.04 to SW 01.05 and amendments to User's Manual version 2.0

## Revision History

Version	Date	Content
1.0	2019-08	New document, created for SW 01.05 and as an amendment to User's Manual version 2.0 for SW 01.04.

### **Purpose of document**

This document provides a description of changes to the User's Manual version 2.0 for SW 01.04.

This supplement is valid for and intended to be used in conjunction with the CoaguChek INRange User's Manual version 2.0. It is not a replacement for the complete CoaguChek INRange User's Manual version 2.0.

Be sure to read the complete CoaguChek INRange User's Manual version 2.0.

---

© 2019 Roche Diagnostics GmbH

The contents of this document, including all graphics, are the property of Roche Diagnostics. No part of this document may be reproduced or transmitted in any form or by any means, electronic or mechanical, for any purpose, without the express written permission of Roche Diagnostics. Roche Diagnostics has made every reasonable effort to ensure that all the information contained in this supplement is correct at the time of printing. However, Roche Diagnostics reserves the right to make any changes necessary without notice as part of ongoing product development.

Please send questions or comments about this supplement to your local Roche representative.

COAGUCHEK and SOFTCLIX are trademarks of Roche.

CONTINUA, the CONTINUA Logos and CONTINUA CERTIFIED are trademarks, service marks, or certification marks of the Continua Health Alliance. CONTINUA is a registered trademark in some, but not all countries this product is distributed.

The *Bluetooth*<sup>®</sup> word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Roche is under license. Other trademarks and product names are the property of their respective owners.

## What is new in SW 01.05.xx?

About (manufacturing information and data encryption key) - BLE device address ..... 5

## Amendments to User's Manual version 2.0


Revised sections .....	7
Performing a test .....	7
Displaying reports on a computer .....	9
Security settings in the Internet browser .....	10

## What is new in SW 01.05.xx?

### About (manufacturing information and data encryption key) - BLE device address

The following information has been added to page 114 with SW 01.05.xx:

In case of technical problems the firmware version is an important piece of information for Roche Diagnostics customer support. Sometimes problems can be solved by applying a firmware update. The **About** screen provides such information.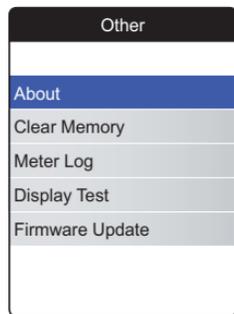

- 1 Select **About** and press the Enter button  to open the corresponding menu.

The serial number, firmware version and the date of the initial setup are displayed.

- 2 Press the Enter button  to switch to the next screen displaying the data encryption key and the BLE (*Bluetooth*<sup>®</sup> Low-Energy) device address.
- 3 Press the Back button  to exit this screen.
- 4 Press the Back button again to go back to the **Other** menu.

The encryption key can also be found encoded as a 2D barcode located in the battery compartment of the meter (batteries have to be removed).

## Amendments to User's Manual version 2.0

### Revised sections

#### Performing a test


The following information has been **added** on page 46:

- 8 Hold the test strip so that the lettering, the arrows and the blood drop symbols are facing upward.
- 9 Slide the test strip into the test strip guide in the direction indicated by the arrows.

Slide the test strip in as far as it will go.

A beep tone indicates that the test strip has been detected (**unless you have disabled the beeper function**).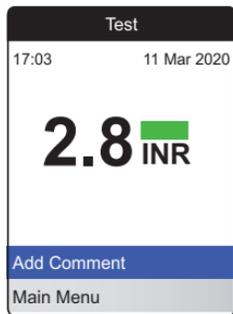

The following information has been **added** on page 51:

The result is displayed in the unit you chose when setting up the meter. It is automatically saved to memory.

When interpreting your result, please also read the test strip package insert carefully.

**If a blue control bottle icon  is displayed along with the result:**

This may occur if the hematocrit value is very low or due to erroneous blood collection (e.g., wet hands). Make sure your hands are dry. Repeat the test. If the icon is displayed again, please contact your healthcare professional or seek immediate medical attention.

## Displaying reports on a computer

The following information has been **added** on page 72:

- 8 After you have finished using the embedded reports, unplug the USB cable from the meter and the computer.
- 9 Your Internet browser may store local copies of the data that is visualized with the help of the embedded reports feature. Therefore, we recommend to empty the Internet browser cache every time after using this feature. In most cases, this will erase any traces of locally stored data.

When the meter is connected to a computer, it will **not** be powered via the USB cable and will **not** power off automatically. To save battery power always remove the USB cable when the connection is no longer in use.

## Security settings in the Internet browser

The following information has been **added** on page 73:

The settings of your Internet Browser may affect your use of reports.

If you choose the security settings appropriately (e.g. "Allow active content to run files on My Computer" or similar), you can work with reports without any restrictions.

**We recommend to empty your browser cache every time after using the embedded reports feature. In most cases, this will erase any traces of data that your Internet browser may have stored locally without your knowledge.**

This page intentionally left blank.

This page intentionally left blank.

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©2019 Roche Diagnostics GmbH